PERFIL TÉCNICO

ISO 4049

ATOS BLOCK

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Indústria Brasileira - Made in Brazil

Marca SMART DENT® - Rev: 00

 **SMART DENT**

SAC +55 16 33743950 | www.smartdent.com.br
sacdentistica@smartdent.com.br

TECHNICAL PROFILE

ISO 4049

ATOS BLOCK

Nano-Hybrid Composite for CAD/CAM

 SMART DENT

SUMMARY

Introduction	3
Main Technological Features	4
BPA FREE	4
Nano-hybrid	4
High Aesthetics	4
High Color Stability	5
Resistance	5
Color and Opacity	5
Main Advantages	6
Indications for Use	6
Color Selection	7
Colors (Two degrees of Translucency)	7
Presentation	7
Composition	7
How to Use	7
Physicochemical and Mechanical Properties	8
Compressive strength	8
Flexural strength	9
Modulus of elasticity	9
Color stability to staining	10
Bond strength to zirconia	11
Bond strength to dentin	12
Color stability	13
Specialist Report Clinical	14
Cases	15

INTRODUCTION

The world's 1st BPA Free CAD/CAM Composite! It's Brazilian! It's Smart!

Atos Block is a nano-hybrid restorative composite for permanent single-unit restorations, which is already polymerized and presented in the form of blocks for milling in dental CAD/CAM systems. The product is fluorescent, radiopaque and contains 77 to 86% by weight of inorganic filler particles in a highly polymerized monomeric matrix that is free of Bisphenol A (BPA Free). The Atos Block composite block has a high degree of conversion, significantly higher than conventional light-cured composite resins. These factors, combined with the adequate particle size distribution, result in a material with high hardness, mechanical strength and wear resistance, allowing the production of parts with an excellent finish, smooth surfaces and high gloss.

Atos Block therefore provides high aesthetics for natural teeth, adjusts the harmony of the smile, as well as restores masticatory functions and contributes to the well-being of dental patients.


MAIN TECHNOLOGICAL FEATURES

BPA FREE

The organic matrix of this restorative is formed by the main structuring monomers in the development of composites in the world, they are manufactured in the USA and have a high molecular weight. Smart Dent launches the world's first composite for CAD/CAM milling with Bisphenol A-free monomers (BPA Free).

NANO-HYBRID:

Its inorganic composition is composed of high-performance silanized fillers manufactured in Germany, such as Barium-Aluminum glass in the range of 0.4 and 0.7 μm , Silica nanoparticles in the range of 35 nm, and mainly Zirconium Nano Silicate in the range of 100 nm, which maximize wear resistance. The uniform distribution of the particles contributes to high mechanical properties, excellent machinability, high aesthetics and especially shine when polished.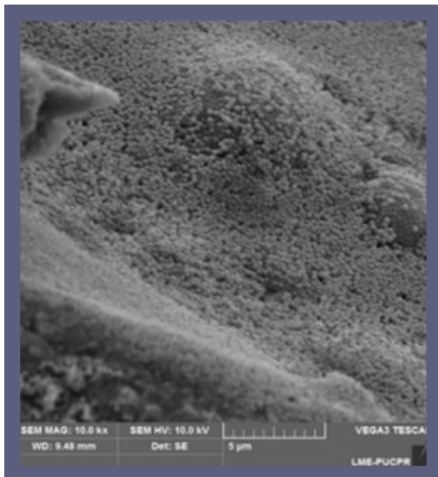

HIGH AESTHETICS:

The spheroidal shape, the size of 100 nanometers and the added percentage of the Zirconium Nano Silicate filler, bring to the restorative composite the power of high brightness and surface smoothness that is visually close to that of a feldspathic ceramic and with prolonged polishing retention.

Scanning Electron Microscopy (SEM) photomicrograph of the spheroidal particles of the ATOS BLOCK restorative composite. **Magnification of 10,000 times.**
Image kindly provided by PUCPR

COLOR STABILITY


The manufacture of the Atos Block composite in block format in industrial ovens with high control of time, temperature, pressure and humidity, means that the blocks have a high degree of conversion of the formed polymer, ensuring its color stability and protecting it from staining. The DELTA E is below 1.0 point, that is, the material does not present any visible color change.

HIGH RESISTANCE

The combination of nanometric particles with micrometric particles in the formulation reduces the space between the filler particles in the monomeric solution, increasing the percentage of filler that, inserted into the structuring monomers, raises considerably improves the physical and mechanical properties of the resin, providing high-strength, long-lasting restorations with the best possible aesthetic results.


Photomicrograph in Electron Microscopy of Scanning (SEM) of the resin particle combination. ATOS BLOCK presents uniformity and does not present discrepant (large) particles. **Magnification of 10,000 times.**

Image kindly provided by FOP-Unicamp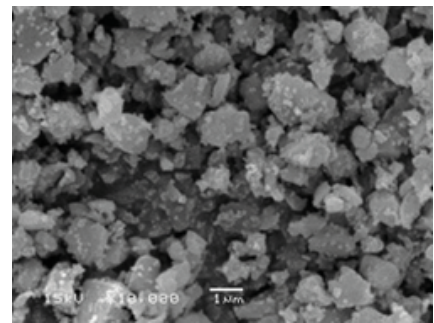

COLORS AND OPACITIES:

Dentin or low translucency (LT) shades exhibit greater opacity and a more saturated chroma, while enamel or high translucency (HT) shades have greater translucency, adjusted to diffuse the dentin substrate. The shades for whitened teeth were developed based on recommendations from dental surgeons specializing in aesthetics. All shades are radiopaque. Atos Block has 11 shade options that meet the needs of the simplest to the most complex aesthetic restorations.

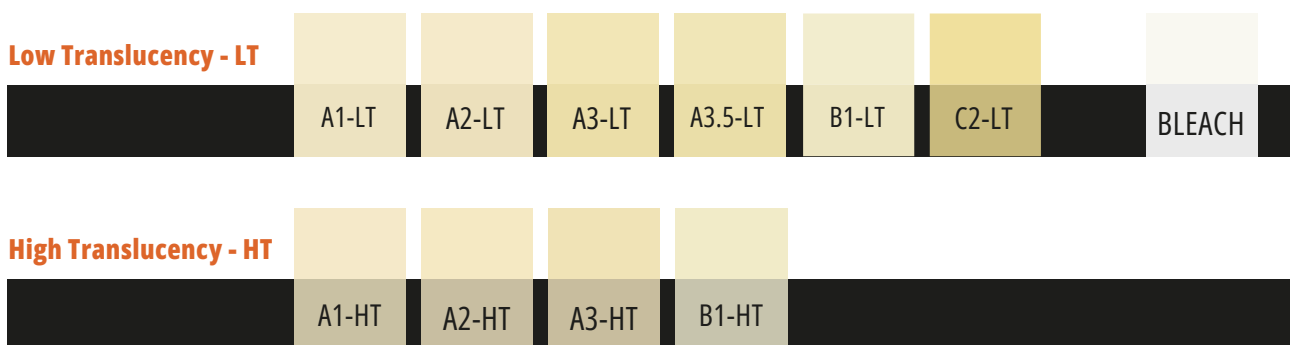NOTE: The block images are illustrative.

MAIN ADVANTAGES

- **Favorable to the antagonist tooth** because the elastic modulus of the composite is similar to that of the tooth;
- **Milling of elements with margins and edges with high precision**, without fractures or lascagens;
- **It does not require firing** (necessary for ceramic blocks, for example) as the composite is already in the desired final color;
- **Allows you to create structures as thin** as 0.4 to 0.6 mm thick for cases of indirect facets;
- **Capable of applying makeup and creating effects** with light-curing composite resins (e.g. Acts);
- **Quick and easy repair power (ceramic material chips have to be** exchanged or repaired with laborious techniques, a possible defect in a composite resin restoration simply requires slightly wearing down the surface at the site of the defect, applying the adhesive and applying composite resin for repair;
- **Greater compatibility with resin cements:** as it is a material with chemical composition compatible with these cements, provides excellent adhesion index;
- **Chairside application (in the clinic) or inLab (in the laboratory);**
- **Compatible with all milling machines** available on the market that have a composite machining strategy.

INDICATIONS FOR USE

Atos Block is indicated for permanent indirect restorations:

- Single full crown
- Single full crown cemented on abutment
- Inlays and Onlays
- Veneers

COLOR SELECTION:

Select the shade using the VITA® shade guide as a reference guide, placing it close to the clean, unprepared tooth and preferably under natural light.

COLORS (TWO DEGREES OF TRANSLUCITY):

High Translucent (HT): A1, A2, A3, B1

Low Translucent (LT): A1, A2, A3, A3.5, B1, C2, BL

PRESENTATION

Kit with 5 units, size 14L, universal holder.

COMPOSITION

Matrix with methacrylate monomers, initiator system, stabilizers, pigments and inorganic fillers composed of Barium-Aluminum glass, Silica nanoparticles and mainly Zirconium Silicate.

HOW TO USE

The dentist will mold or scan the patient's oral cavity, then send it to the laboratory (inLab) and/or design the piece in dental CAD software and mill the block (Chairside) according to the intended restorative treatment. The machining strategy to be chosen in CAM should be composites.

Restorations are cemented to the tooth structure or abutment with dental adhesive (containing MDP) and dental resin cement. The steps for manufacturing CAD/CAM systems are described in the instructions for use and manuals of each manufacturer. Manufacturers' recommendations should be followed.

See the Atos Block User Instructions.

PHYSICOCHEMICAL AND MECHANICAL PROPERTIES

Tested by **FACPP** and **FOP-UNICAMP** universities

COMPRESSION STRENGTH

The mechanical property of compressive strength demonstrates the ability of the restorative composite to withstand vertical stresses, since the forces transmitted to the restorations during the act of mastication can trigger fractures of the material. The higher the value of compressive strength, the greater the longevity of the composite.

The mechanical compression test was performed on the EMIC-Instron universal testing machine (Model 23-2S, São José dos Pinhais-PR Brazil), with a load cell of 50,000 Kgf and speed of 1 mm/min. EMIC indicated the total breaking strength of the test specimens. For this purpose, a bluhill version 2.0 program was used (EMIC, São José dos Pinhais - PR, Brazil), which archives the results in a database in the form of graphs and numerical values in MPa.

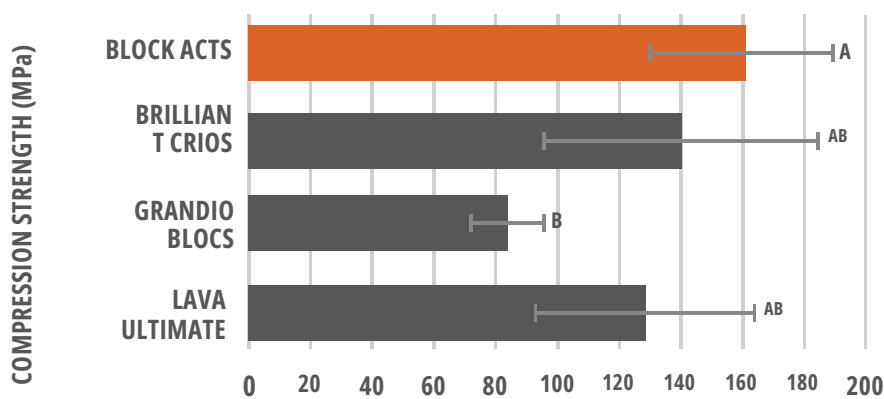
Figure 1: Compressive strength (mean and SD in MPa) in 4 different composites (n=5). Static ANOVA analysis and Tukey test; $p < 0.05$.

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP, Paulo Picanço College of Fortaleza (FACPP) and School of Dentistry of Piracicaba (FOP-Unicamp) 2022.

The **ATOS BLOCK** composite presents **AMAZING COMPRESSION STRENGTH VALUES**, being **HIGHER THAN ALL COMMERCIAL BRANDS** tested of restorative composites for CAD/CAM. Statistically, by standard deviation, the results are similar for Lava Ultimate (3M) and Coltene (Brilliant Crios), Grandio Blocs (VOCO) presented lower compressive strength results.

FLEXIBLE STRENGTH

Flexural Strength is one of the most important mechanical tests for composites dental as it evaluates the fracture resistance of the restorative material as well as its intrinsic properties (monomeric matrix, inorganic fillers and initiators), in relation to different masticatory forces.

The test was carried out in accordance with ISO 4049:2019, using a universal testing machine.

EMIC-Instron (Model 23-2S, São José dos Pinhais-PR Brazil), the blocks were machined to specific dimensions (18x2x2x mm) in rod shape. Five specimens were manufactured from each block. The specimens were subjected to a three-point bending strength test.

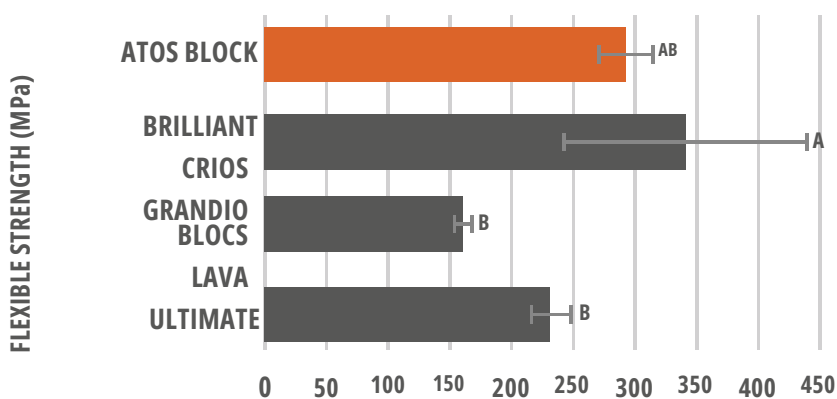
Figure 2: Flexural strength (mean and SD in MPa) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; $p < 0.05$.

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP. Paulo Picanço College of Fortaleza (FACPP) and School of Dentistry of Piracicaba (FOP-Unicamp) 2022.

BLOCK ACTS presents **EXCELLENT FLEXIBLE RESISTANCE**, comparable to well-known products on the market, being statistically equal to Coltene and superior to VOCO and 3M blocks.

ELASTICITY MODULE

The term modulus of elasticity or flexural modulus describes the rigidity of the resin and is important for determining the resistance to occlusal forces and wear resistance, which reflects an intrinsic property of the material to resist pressures imposed by masticatory forces. The higher the elastic modulus, the greater the resistance of the composite to fractures.

The blocks were machined to specific dimensions (18x2x2x mm) in rod shape. 5 specimens were made from each block. The specimens were subjected to a three-point modulus of elasticity test in a universal testing machine.

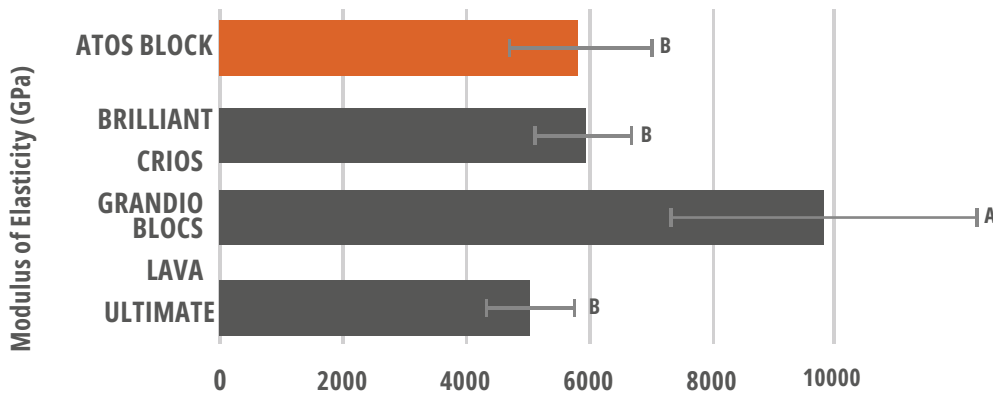
Figure 3: Flexural strength (mean and SD in MPa) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; $p < 0.05$.

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP. Paulo Picanço College of Fortaleza (FACPP) and School of Dentistry of Piracicaba (FOP-Unicamp) 2022.

It can be seen that **ATOS BLOCK** presented elastic modulus comparable to international products already known on the market.

COLOR STABILITY TO STAINING

Staining in dental composites can occur due to the ability of the restoration absorb or adsorb pigments from the oral cavity, which can be enhanced by the physical-chemical structure conditions of the raw materials that make up the material. The samples of the 7x7 mm and 1 mm thick blocks were polished and the initial color evaluation was obtained using the Vita Easyshade spectrophotometer (Wilcos) considering the values of "L" and "E" as a numerical unit and the color scale provided by the equipment. After 7 days immersed in black tea (with a proportion of 250 mL to 18 grams) at 37 °C in an oven, a new color analysis was measured and then the color difference was calculated.

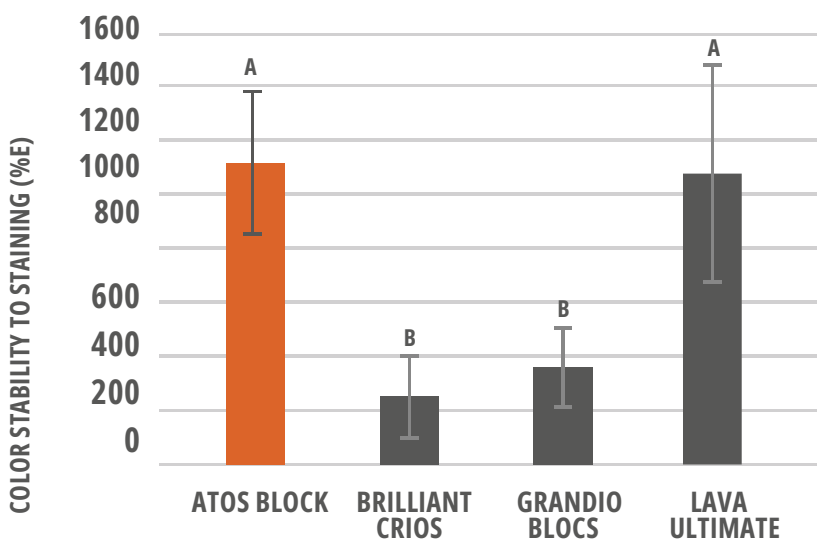
Figure 4: Color Stability to Staining (mean and SD in %) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; $p < 0.05$.

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP. Paulo Picanço College of Fortaleza (FACPP) and School of Dentistry of Piracicaba (FOP-Unicamp) 2022.

The **ATOS BLOCK** and Lava Ultimate (3M) blocks presented the **BEST RESULTS** in terms of color stability to staining, statistically different for Grandio Blocs (VOCO) and Brilliant Crios (Coltene) which suffered more color change due to staining.

ZIRCONIA ADHESIVE STRENGTH

Zirconia (Zr) is one of the most widely used ceramic materials in the world for prosthetic rehabilitation, mainly because it has excellent mechanical strength and biocompatibility. However, it is known from technical and literary sources that Zr is the most complex material in terms of adhesion, being extremely difficult to bond with and easy to debond. Therefore, if the restorative material adheres to Zr, the material meets the adhesion requirements to any other substrate (such as dentin and enamel, metals and other ceramics).

The Zr specimens (4 mm x 4 mm) received an application of phosphoric acid for 30 s followed by two layers of adhesive with a light air jet of 3 to 5 s between them, photopolymerized by LED (Valo, Ultradent, South Jordan, USA) with irradiance of 1200 mW/cm² for 40 s. Tube-shaped test specimens of Relyx ARC cement (3M Oral Care, St. Paul, MN, USA) were adhered to the specimens, cementing the specimens of each block.

The microshear bond strength was performed on a universal testing machine (EMIC-Instron, São José dos Pinhais, Brazil) and the values calculated in MPa.

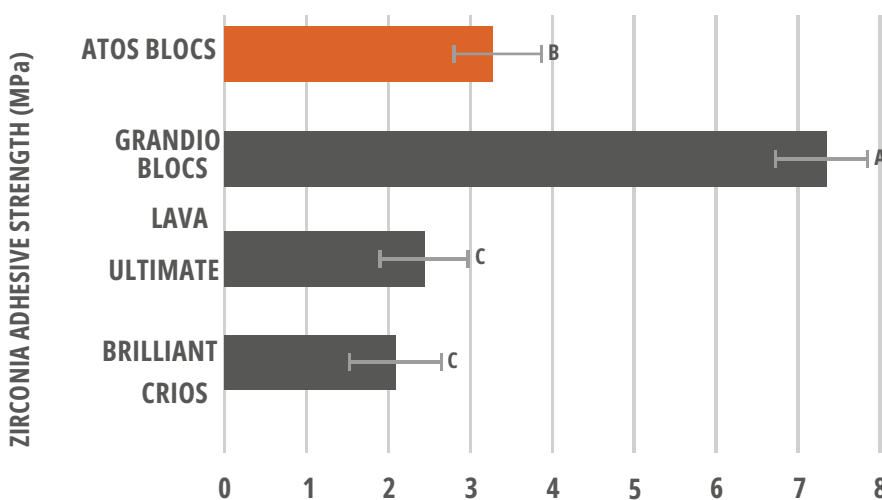
Figure 5: Bond Strength to Zirconia (mean and SD in MPa) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; p<0.05

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP. Paulo Picanço College of Fortaleza (FACPP) and Piracicaba School of Dentistry (FOP-Unicamp) 2022

ATOS BLOCK and Grandio Blocs presented **SUPERIOR ADHESION TO ZIRCONIA** than **ALL THEIR COMPETITORS**. The test revealed that **ATOS BLOCK** was remarkable, even though Zr is a material that is extremely difficult to accept adhesion. Given its efficiency in Zr, Atos Block is effective on any substrate as well as on dentin and enamel, on metals and other ceramics!

ADHESION STRENGTH TO DENTIN

The clinical success of a restoration is based mainly on the adhesion strength provided by the adhesive system. One of the most commonly used tests to study adhesive bond strength is the tensile test. In restorations that use composites and adhesive systems, optimal adhesion will depend on the material's ability to withstand immediate and ageing mechanical stresses resulting from its curing mechanism and its physical-mechanical behavior.

Twenty extracted third molars were used, where each tooth was cut to obtain specimens on flat dentin discs. Each specimen was sanded for 60 seconds to obtain the standardized smear layer. The CAD/CAM blocks were cut into bars measuring 1x1x3 mm. Universal adhesive was applied and light-cured with Radies Xpert (SDI, AUSTRALIA). All resin pieces were silanized and cemented with (Relyx ARC, 3M, SÃO PAULO) and light-cured for 40 seconds. The use of materials followed the manufacturers' recommendations. After 24 hours of immersion in distilled water, the teeth were sectioned on resin-dentin sticks (1 mm² section area). The bond strength test was performed on a universal testing machine EMIC 23-2S (EMIC-Instron, São José dos Pinhais, Brazil) tested under tensile force, at a speed of 0.5 mm/min, with a load cell of 2000 N, until the specimen ruptured.

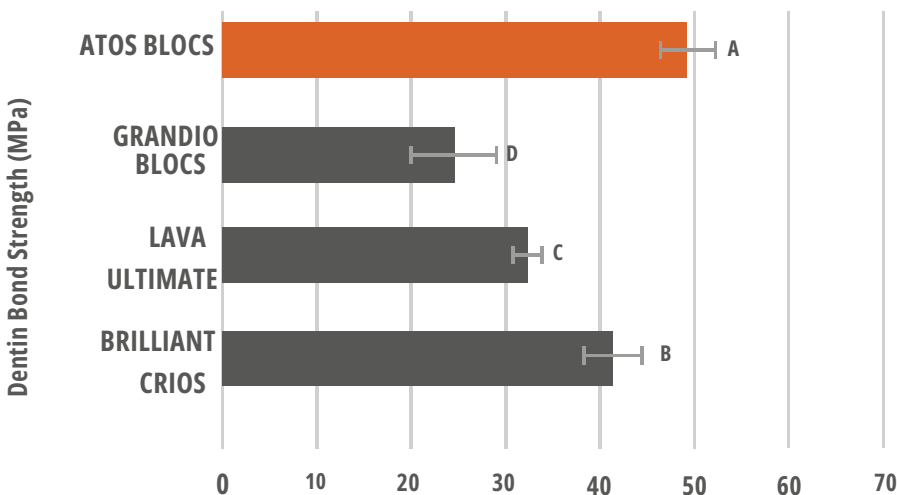
Figure 6: Dentin Bond Strength (mean and SD in MPa) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; p<0.05.

Source: Rifane TO, Cordeiro KE, Silvestre FA, Alves AHC, Nobre CFA, Araújo-Neto VG, Giannini M, Feitosa Faculty of Dentistry of Piracicaba (FOP-Unicamp) 2021.

The **ATOS BLOCK** composite showed **ADHESION** to the **DENTIN UPPER** than **ALL ITS COMPETITORS**. The test shows that the **ATOS BLOCK IS REMARKABLE AND EFFECTIVE** on this substrate, bringing **GREATER SAFETY TO THE CLINICIAN AND THE PATIENT**.

COLOR STABILITY

Among the characteristics related to restorative materials, color stability has been one of the most important clinical parameters for the success of an aesthetic restoration. Color changes in dental composites, in the medium or long term, compromise aesthetic restorations, making replacement of the restoration a frequent phenomenon in clinical practice.

The specimens of the blocks measuring 7x7 mm and 1 mm thick were polished and the initial color evaluation was obtained using the Vita Easyshade spectrophotometer (Wilcos). After 5 days in an oven at 80°C, a new color analysis was measured and then the color difference was calculated.

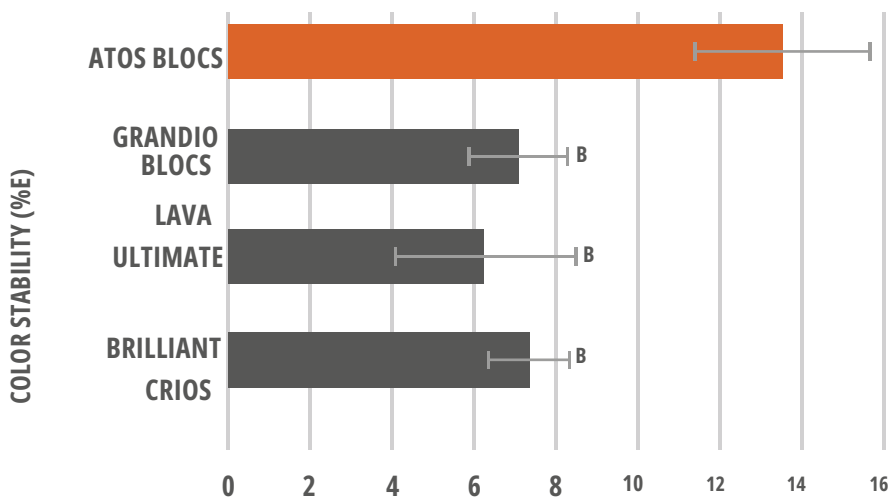
Figure 7: Color stability (mean and SD in %) in 4 different composites (n=5). Statistical analysis ANOVA and Tukey test; p<0.05.

Source: Silvestre FA, Kelly K, Moreira ACL FA, Feitosa VP. Paulo Picanço College of Fortaleza (FACPP) and Piracicaba School of Dentistry (FOP-Unicamp) 2022

ATOS BLOCK presented the **BEST COLOR STABILITY RESULTS**, statistically different to Grandio Blocs (VOCO), Lava Ultimate (3M) and Brilliante Crios (Coltene) which underwent more color changes.

EXPERT REPORTS

Atos Block has **excellent mechanical properties** that will amaze dentists. Parts made from this material have the reliability of a very well-cured resin in an easily milled block for **long-lasting aesthetics and strength.**

PROF. DR. VICTOR FEITOSA – FORTALEZA - CE

PROFESSOR AT FACPP AND UNIFAMETRO - FORTALEZA-CE, DENTIST, MASTER AND DOCTOR FROM FOP - UNICAMP, POSTDOCTORATE IN DENTISTRY FROM UFC.

The ATOS BLOCK restorative composite has **excellent physical-chemical and mechanical properties**, surpassing international standards as it has the best in science and **technology of dental materials. It also stands out for its incomparable color stability and its impressive shine when polished.**

PROF. DR. RONY PETERSON RODRIGUES – PIRASSUNUNGA - SP

TPD, TEC QUIM, ENG. PROD, ENG. MAT, DOCTOR (USP)

What caught my attention in the Atos Block was the **shine we achieved with polishing. Super practical for Chair Side restorations!**


PROF. DR. VIVIANE LACERDA - RIO DE JANEIRO - RJ

DENTAL PROSTHETICS SPECIALIST, MASTER IN DENTAL PROSTHETICS, MASTER IN DIGITAL DENTISTRY

CLINICAL CASE

AESTHETIC AND FUNCTIONAL REHABILITATION OF TEETH IN DIGITAL FLOW

Author: Prof. Dr. Eduardo Mukai – Master in biomaterials from USP (FOB), PhD in dental prosthetics from USP (FO)

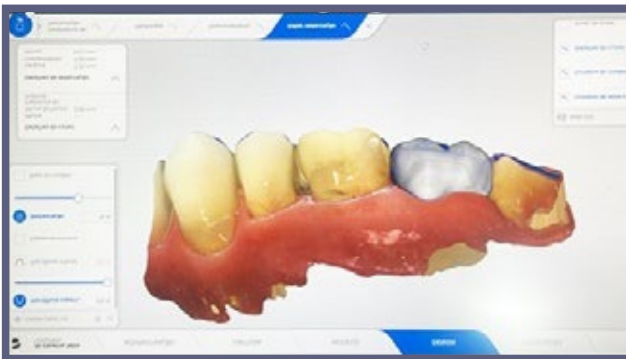1. Intraoral scanning and planning of the prosthetic crown

2. Planning the prosthetic crown to be machined in Atos Block

3. Machined and polished prosthetic crown.

4. Cemented prosthetic crown – final clinical case


“Milling Atos Block was easy, very smooth, very similar to international composite blocks but with a **more interesting color stability!**”

PROF. DR. EDUARDO MUKAI – SAO PAULO - SP

AESTHETIC REHABILITATION WITH CAD/CAM COMPOSITE MILLING


Author: Dr. Carlos Eduardo Pulici – Master in dental prosthetics, specialist in implantology, dental prosthetics and restorative dentistry.

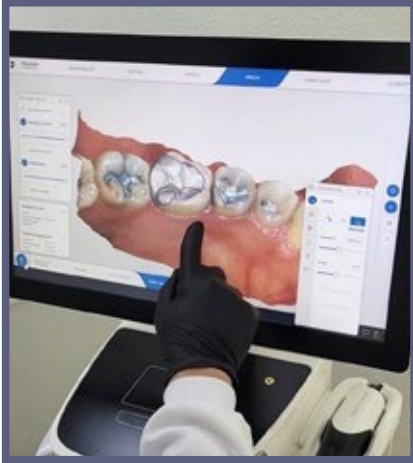1. Digital strategic planning of the clinical case

2. Removal of the decayed area of the tooth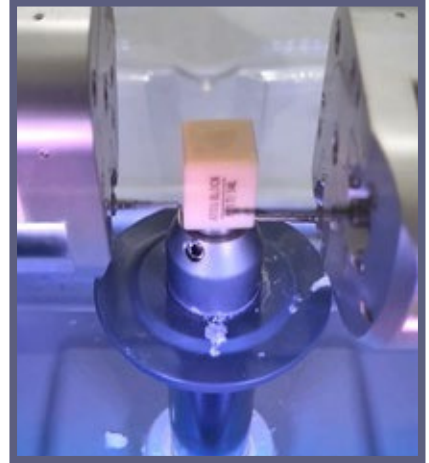

3. Start of machining of Atos Block by CAD/CAM milling machine


4. Machined onlay

5. Prosthetic rehabilitation, tooth 36 cemented with Atos Block.

SMART DENT's Atos Block is an excellent material for making restorations in the CAD/CAM system, combining **aesthetics and functionality**, resulting in **high quality and precision work**.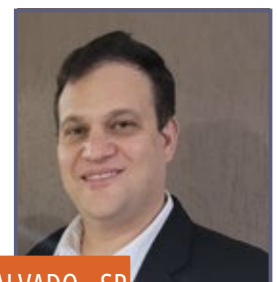

DR. CARLOS EDUARDO PULICI – DESCALVADO - SP

SMART DENT


ALSO KNOW

ATOS is a light-curing nano-hybrid composite resin, indicated for restorations of all classes in anterior and posterior teeth using the incremental system.

ATOS UNICHROMA is based on the **latest technology in mimicry**. If there are surrounding walls, Atos Unichroma mimics the color of the tooth. Due to its excellent refractive index with color scattering, it is possible to copy the dental substrate.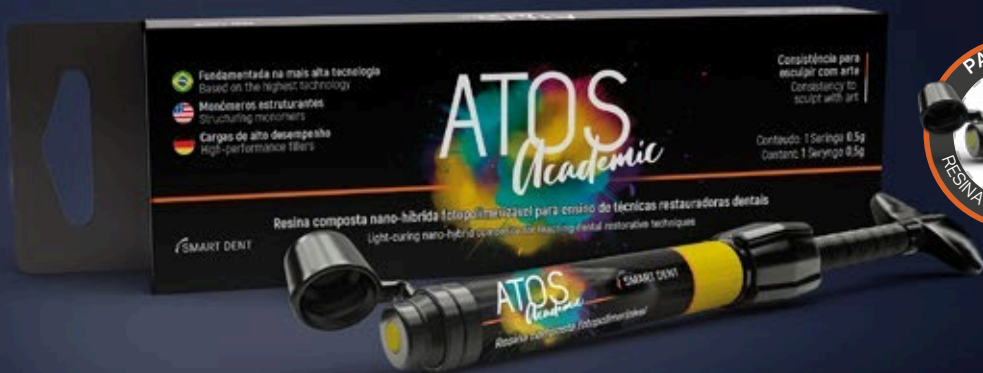

ACADEMIC ACTS It is the colored version of Atos resin, made specifically for teaching dental restorative techniques.

SMART ORTHO is an orthodontic adhesive, developed for direct bonding of metal, ceramic and polycarbonate brackets to the dental surface.

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