ATOS

MMTech LTDA

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Professional Use Only

Before using the product, read all the information into the instruction manual. Keep it for later consultation until the total consumption of the product and/or until there is no more interaction of the product with your last patient.

Product indication

- Restorations of permanent and deciduous teeth, anterior and posterior, classes I, II, III, IV and V [all classes]
- Composite resin direct veneers
- Bonding of tooth fragments
- Anchoring teeth [splinting]
- Correction of teeth shape [ex: conoids]
- Reduction or closure of diastemas
- Making filling cores
- Non-carious cervical lesions
- Porcelain and/or composite repairs
- Indirect restorations, such as inlays, onlays

and facets

- Cementation of prosthetic parts [according to the thermo-modified resin technique]

Product description

Atos resin is a photopolymerizable dental restorative material, called universal composite resin; indicated for restorations of all classes in anterior and posterior teeth by the incremental system. The composite is nano-hybrid and has fluorescence and opalescence. Atos is radiopaque, its inorganic charge is composed of Barium-Aluminum glass and nanoparticles of Silica and mainly Nano Zirconium Silicate. The average particle size is 0,3 microns, with silanized particle size ranging from 35 nm to 1,0 µm. characteristics that contribute to high mechanical properties and high aesthetics, especially polishing shine. The total content of inorganic filler by weight for Atos resin is 78 % to 82 % [57 % to 61 % by volume]. Atos resin is composed of a matrix with dimethacrylate monomers of BisGMA, BisEMA, UDMA, DDDMA and TEGDMA.

Like all types of photopolymerizable dental composites, Atos resin makes use of a polymerization system by means of photoactivation, that is, it uses blue light [in the range of 468 nm] halogen or LED to promote its polymerization.

In this way, it is possible to polymerize the material after working on the proper positioning of composite resin and the shape of the restoration.

The range of colors and variations of opacities were developed and based on the highest mimic technology for dental aesthetics, allowing success in the stratification technique, in which together with the other characteristics, provide maximum aesthetic performance and mimicry capacity of natural teeth.

The translucent OPL color has a high opalescent optical effect and therefore this color is radioluculent.

Therefore, Atos resin allows aesthetics to natural teeth as well as restoring masticatory functions and contributing to the well-being of dental patients.

Product use instruction

Color selection:

For the color selection, make a prophylaxis. The teeth must be clean and hydrated, without isolation, as dehydration can change the color.

Methods for choosing the color:

- Can be performed with the VITA Classical® shade guide commonly used;

- Alternatively, it can be made with a small portion of composite resin which must be placed on the buccal surface of the tooth to be restored [without conditioning and application of an adhesive system], polymerized for 30 seconds and moistened with the patient's saliva. This step can be performed to confirm the color of the composite resin;
- For the most challenging cases, we recommend that, initially, the professional make the chromatic map of the tooth. The chromatic map is all the more detailed information of the individual characteristics of the teeth.

Product Application

1. Isolate the operating field. The insulation must ensure a clean, visible and moisture-free operating field throughout the treatment. Preferably, make absolute isolation;
2. Make the preparation for the execution of direct adhesive restorations;
3. Perform acid conditioning, as reported by the product protocol used;
4. Apply the adhesive system according to the protocol of the product used;
5. Insertion and Light Curing of Composite Resin.
- We recommend the incremental insertion and polymerization technique. This technique allows a polymerization more uniform. Use the polymerization times described in Table 1 for layers up to 2,0 mm thick of resin.
- Restore the cavity according to the selected core(s) by applying small layers (maximum thickness of 2,0 mm) carefully adapting the cavity.
- The OPL color should only be used as an intermediate layer in the incisal region.
- To allow a better polymerization of the last resin layer, light cure for 40 seconds.

NOTE:

- [1] Use photopolymerizer with 400 to 500 mW with power from 600 mW/cm².
- [2] Checking the occlusion is an extremely important factor. Supracclusion restorations cause great stress on the tooth and restoration, which can lead to pain.
- [3] Sanding discs or fine-grained drills can be used for finishing, and for polishing, felt discs and polishing pastes are recommended.

Shades

Enamel: EA1, EA2, EA3, EA4, EB1 and EB2. For

bleached teeth: WE and XWE
Dentina: DA1, DA2, DA3, DA3.5, DA4, DB1 and DB2. For bleached teeth: WD and XWD
Effect: Opaque, Translucent Clear and Translucent Opalescent OPL.

Definition of colors/acronyms:

- E = Enamel and D = Dentina, for traditional colors [VITA Classical® shade guide]
- W = White and XW = Extra White (for bleached teeth)
- Opaque to mask darkened substrates
- Clear for Palatal Shell and Incisal area
- OPL = Opalescent, optical effect color for enamel (with incident light it becomes bluish, with transmitted light its color changes to orange).

Table 1 : Light- Curing Time		
Color	Thickness	Time
Dentin	2 mm	30 - 40 s
Enamel	2 mm	20 - 30 s
Translucent	2 mm	20 s
Opaque	2 mm	50 s

Note:
The longer specified time gives the formed polymer greater properties.

Presentation in single dose capsule [0,2g]:

In addition to the traditional syringe presentation, Atos Caps resin is available as Unidose Capsules for use in a Centrix syringe. To use this system, simply attach the capsule to the syringe, remove the cap from the capsule and press the trigger of the syringe which will extrude the resin into the tooth. After use, even if the content is not completely used, it must be properly discarded.

Presentation forms

- Printed on the product packaging
- Package containing 1 syringe with 2 or 4 g of the product and instructions for use for the professional.
 - Package containing 2 or more syringes with 2 or 4g of the product and instructions for use for the professional.
 - Kits containing syringes with 2 or 4 g of the product and instructions for use for the professional.
 - Package containing 1 to 20 capsules with 0,2 g each.

Basic composition:

Contains methacrylate monomers,

stabilizers, photoinitiator, co-initiator, fillers, silane and pigments.

Expiration date:

Printed on the product packaging.

Storage conditions:

The product must remain in its original packaging.
Keep the product in a cool place with the packaging always tightly closed.
Protect from direct sunlight.
Store the product at temperatures from 5 °C to 27 °C / 41 °F to 80 °F.
Do not freeze the product.
Do not use the product after the expiration date.
If necessary, with the aid of alcohol, remnants of the unpolymerized product can be cleaned.

Handling Conditions

- The product is indicated for professional use only.
- Read the instructions and instructions for use carefully.
- Do not use the product if it is out of date.
- To dispose of the product, follow the legislation of your country.
- Clean instruments with alcohol after use.
- Put the cap back on the syringe after each use.
- The use of gloves is recommended.
- The capsule presentation is for single use, discard the capsule after use.

Warnings

Do not use the product if it is out of date.
Keep out of the reach of children.

Precautions

Professional use only

- This product is indicated only according to its instructions for use.
- Avoid the use of eugenol based lining or provisional materials, as these interfere with the polymerization of the material.
- Avoid contact of the unpolymerized product with the skin, mucosa and eyes.
- Avoid use in patients with a history of allergy to composite resins based on methacrylates. In case of allergy, discontinue use and refer the patient for medical assistance.
- When using a light curing device, eye protection must be used.
- Sensitive to light. Always keep the product closed when it is not in use. After extruding the product out of the syringe or capsule, the contents should be used quickly.
- Light curing equipment must have its power checked with a radiometer frequently.

Contraindications and adverse effects

The product is contraindicated for patients

allergic to methacrylates. The product contains methacrylate monomers that can cause mild allergic or irritant reactions in patients sensitive to these substances. In case of allergic reactions to the product, use must be suspended.

This material has been manufactured for dental use only and must be handled, according to the instructions for usage. The manufacturer is not responsible for any damage caused by improper use or manipulation of the product. The user is obliged to check, before use and under his responsibility, if this material is compatible with the intended use, especially when it is not indicated in this instruction for use. Data descriptions are not a guarantee of any kind and therefore are not binding.

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