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Professional Use Only

Before using the product, read all the information into the instruction manual. Keep it for later consultation until the total consumption of the product and/or until there is no more interaction of the product with your last patient.

Product description

Atos Unichroma composite resin is a light-curing dental restorative composite, with a single color and opacity, for all dental shades, called unichromatic resin (universal color); indicated for restorations of all classes in anterior and posterior teeth by the incremental or bulk fill system.

The composite is nano-hybrid and has fluorescence, opalescence and radiopacity. Its inorganic filler is composed of Barium-Aluminum glass, Silica nanoparticles and Zirconium Nano Silicate. The average particle size is 0.3 microns, with a silanized particle size range from 35 nm to 0.7 µm, characteristics that contribute to high mechanical properties and high aesthetics, especially polishing gloss. The total inorganic filler content by weight for Atos Unichroma resin is 74% to 80% (56% to 61% by volume).

Atos Unichroma resin is composed of a matrix with structuring dimethacrylate monomers of BisGMA, BisEMA, UDMA, DDDMA and TEGDMA. Its polymerization system is by means of photocatalysis with halogen blue light or LED. In this way, it is possible to polymerize the material after working the proper positioning of the composite resin and the format of the restoration. It has low polymerization contraction. No coating resin needed!

This material was developed and based on the highest mimicry technology, allowing success with just a single resin color in a true chameleon effect. With surrounding walls, Atos Unichroma mimics the color of the tooth, due to its excellent refractive index with color scattering, it is possible to copy the dental substrate.

The composite with the aforementioned characteristics provides aesthetic performance, mechanical resistance and color mimicry. Therefore, Atos Unichroma resin allows aesthetics to natural teeth as well as restoring masticatory functions, ease of use and agility

in restorations, and contributes to the well-being of dental patients.

Product indication

- Restorations of permanent and deciduous teeth, anterior and posterior, classes I, II, III*, IV* and V [all classes]
- Composite resin direct veneers
- Bonding of tooth fragments
- Anchoring teeth (splinting)
- Correction of teeth shape (ex: conoids)
- Reduction or closure of diastemas
- Non-carious cervical lesions
- Porcelain and/or composite repairs
- Indirect restorations, such as inlays and onlays

* For restorations with large cavities such as class III and IV (without a back wall to support the color) use an Opacifying resin (e.g.: Atos resin in Opaquer or Dentina A1, A2, etc color) which will serve as a base for restore and finalize the remainder of the volume with Atos Unichroma. This procedure is necessary as Atos Unichroma can appear semi-translucent when not surrounded by the walls of the dental cavity.

Product use instruction

Product description

1. Isolate the operating field. The insulation must ensure a clean, visible and moisture-free operating field throughout the treatment. Preferably, make absolute isolation;
 2. Make the preparation for the execution of direct adhesive restorations;
 3. Perform acid conditioning, as reported by the product protocol used;
 4. Apply the adhesive system according to the protocol of the product used;
 5. Insertion and Light Curing of Composite Resin:
- * Incremental polymerization of 2 mm is 20 s and bulk up to 5 mm is 30 s.
 - * Restore the cavity by applying the material by carefully fitting the cavity.

NOTE:

- (1) Use photopolymerizer with 400 to 500 nm with power from 600 mW/cm².
- (2) Checking the occlusion is an extremely important factor. Supraocclusion restorations cause great stress on the tooth and restoration, which can lead to pain.
- (3) Sanding discs or fine-grained drills can be used for finishing, and for polishing, felt discs and polishing pastes are recommended.

Remarks:

- (1) For porcelain/composite staining or repair,

use an Opacifying resin (e.g.: Atos resin in Opaquer color) or Dentina shade (e.g.: Atos resin in DA1, DA2, etc. color) with layer thickness of 1 to 2 mm to mask darkening. (2) Atos Unichroma will only mimic tooth color if there are surrounding walls. (3) The composite has a slightly higher opacity before curing, which will resemble the tooth color especially after light curing.

Table 1 : Light- Curing Time		
Color	Thickness	Time
Unichromatic	5 mm	30 s
	2 mm	20 s

Presentation in single dose capsule (0,2g):

In addition to the traditional syringe presentation, Atos Caps resin is available as Unidose Capsules for use in a Centrix syringe. To use this system, simply attach the capsule to the syringe, remove the cap from the capsule and press the trigger of the syringe which will extrude the resin into the tooth. After use, even if the content is not completely used, it must be properly discarded.

Presentation forms

Printed on the product packaging

- Package containing 1 syringe with 2 or 4 g of the product and instructions for use for the professional.
- Package containing 2 or more syringes with 2 or 4 g of the product and instructions for use for the professional.
- Kits containing syringes with 2 or 4 g of the product and instructions for use for the professional.
- Package containing 1 to 20 capsules with 0.2 g each.

Basic composition:

Contains methacrylate monomers, stabilizers, photoinitiator, co-initiator, fillers, silane and pigments.

Expiration date:

Printed on the product packaging.

Storage conditions:

The product must remain in its original packaging. Keep the product in a cool place with the packaging always tightly closed. Protect from direct sunlight. Store the product at temperatures from 5 °C to 27 °C / 41 °F to 80 °F.

Do not freeze the product.
Do not use the product after the expiration date.
If necessary, with the aid of alcohol, remnants of the unpolymerized product can be cleaned.

Handling Conditions

- The product is indicated for professional use only.
- Read the instructions and instructions for use carefully.
- Do not use the product if it is out of date.
- To dispose of the product, follow the legislation of your country.
- Clean instruments with alcohol after use.
- Put the cap back on the syringe after each use.
- The use of gloves is recommended.
- The capsule presentation is for single use, discard the capsule after use.

Warnings

Do not use the product if it is out of date.
Keep out of the reach of children.

Precautions

Professional use only

- This product is indicated only according to its instructions for use.
- Avoid the use of eugenol based lining or provisional materials, as these interfere with the polymerization of the material.
- Avoid contact of the unpolymerized product with the skin, mucosa and eyes.
- Avoid use in patients with a history of allergy to composite resins based on methacrylates. In case of allergy, discontinue use and refer the patient for medical assistance.
- When using a light curing device, eye protection must be used.
- Sensitive to light. Always keep the product closed when it is not in use. After extruding the product out of the syringe or capsule, the contents should be used quickly.
- Light curing equipment must have its power checked with a radiometer frequently.

Contraindications and adverse effects

The product is contraindicated for patients allergic to methacrylates. The product contains methacrylate monomers that can cause mild allergic or irritant reactions in patients sensitive to these substances. In case of allergic reactions to the product, use must be suspended.

This material has been manufactured for dental use only and must be handled, according to the instructions for usage. The manufacturer is not responsible for any damage caused by improper use or manipulation of the product. The user is obliged to check, before use and under his responsibility, if this material is compatible with the intended use, especially when it is not indicated in this instruction for use. Data descriptions are not a guarantee of any kind and therefore are not binding.

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