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Professional Use Only

Before using the product, read all the information into the instruction manual. Keep it for later consultation until the total consumption of the product and/or until there is no more interaction of the product with your last patient.

Product indication

Adhesive cementation of indirect veneers in ceramics, composites, 3D resins and composite resins (manufactured in the office or laboratory) with a thickness of up to 1.5 mm that allow the use of only light-curing cement due to the high level of translucency.

Product Description Unikk Veneer LC

Unikk Veneer LC is a light-curing dental resin cement for cementing indirect veneers up to 1.5 mm thick that do not have an opaque infrastructure.

The cement is presented as a single component (single syringe) and should be used according to the adhesive technique (acid etching + adhesive). The cement is radiopaque (allows excess cement to be identified on radiographs) and has excellent color stability. For a more natural visual appearance, the cement has fluorescence compatible with that of dental enamel. The viscosity of Unikk Veneer LC was carefully developed to facilitate the application of the product.

Like all types of light-curing dental composites, Unikk Veneer LC uses a polymerization system through photoactivation, that is, it uses blue light (in the 468 nm range) halogen or LED to promote its polymerization. This allows the material to polymerize after working on the correct positioning of the prosthetic piece.

The opacity and color range were developed and based on the highest mimicry technology for dental aesthetics, allowing for success in the cementation technique, which, together with the other characteristics, provides maximum aesthetic performance and the ability to mimic natural teeth.

The consistency of the resin cement allows the prosthetic piece to be firmly positioned before photopolymerization, which prevents poor positioning during cementation.

The product's attributes provide simplified use and safety in the color selection process, and the test can be performed with the Unikk Veneer Try-in glycerin paste.

Unikk cements have two important characteristics for color matching between the piece and the substrate. The veneer line features a particle arrangement with OLD (Optimized Light Dispersion) technology, where light is widely scattered within the material, allowing a blending effect with the veneer, resulting in highly natural results. The LV paste has PAR (Pressure Adaptive Rheology) technology. Its monomers and particle adjustment provide a change in its viscosity according to the insertion pressure of the pieces, making it less viscous at this stage and returning to its original state when the excess is cleaned.

Product Description Unikk Veneer Try-in

Unikk Veneer Try-in is a pigmented, water-soluble paste designed to allow for color testing of indirect restorations with Unikk Veneer LC resin cement, mimicking the color of the cement that will be selected for definitive cementation. The colors of the pastes faithfully follow the colors of the cement, so that the selected color of the Unikk Veneer Try-in paste provides a result equivalent to that of the final cementation after photopolymerization.

Instructions for use

Before cementation, check whether the characteristics of the veneer are within the cement's indications: translucent piece made of ceramic or resin with a thickness of 1.5 mm or less.

Select the color of Unikk Veneer Try-in paste that matches the color of the restoration. Place Unikk Veneer Try-in inside the prosthetic piece and position it on the tooth. Remove any excess. Check the

chromatic effect generated by the selected paste. If necessary, evaluate other colors to achieve better aesthetic harmony. In this case, remove the prosthetic piece from the tooth, remove the Unikk Veneer Try-in paste by washing with plenty of water/air, dry and try another color of paste. After selecting the color of Unikk Veneer Try-in paste that best suits your needs, wash the prosthetic piece with plenty of water, dry and prepare the piece for subsequent cementation using the respective definitive cement.

Color selection:

To select the color, perform prophylaxis. The teeth must be clean and hydrated, without isolation, since dehydration can change the color..

1. Methods for choosing the color:

- Use Unikk Veneer Try-in pastes to determine the best color of cement for the case. Apply the paste to the prosthetic piece and then to the tooth, pressing gently, to see if it is the desired color. After choosing the color, wash the piece and the tooth with plenty of water, dry with air and gauze to remove any residue of the material.
- This can be done using the VITA Classical® scale commonly used as a guide.
- Pode ser realizada com a escala VITA Classical® comumente utilizada como guia.
- Alternatively, it can be done with a small portion of the cement, which should be placed on the vestibular surface of the tooth (without conditioning or application of an adhesive system), polymerized for 30 seconds and moistened with the patient's saliva. This step can be performed to confirm the color of the cement.

2. Tooth preparation

2.1 Remove the temporary restoration and any temporary cement residue using pumice stone and water with a prophylactic brush.

2.2 Perform absolute isolation or effective relative isolation of the tooth. Perform gingival retraction with retraction cords.

2.3 Apply two layers of adhesive under vigorous friction for 10 s each. Do not light cure.

3. Preparation of the prosthetic piece

3.1 The internal surface of the piece can be sandblasted (e.g., aluminum oxide) to increase microretention (laboratory procedure).

3.2 To condition the veneers, apply hydrofluoric acid for the recommended time according to the product instructions. Wash thoroughly with water and dry the piece.

3.3 When the internal surface of the piece is clean, apply a layer of silane (10 to 15 s) and wait for it to dry for 1 minute.

3.4 Apply two thin layers of adhesive. After application, apply a light jet of air (for 10 s) to evaporate the solvent. Do not light cure.

4. Cementation

After the tooth and the piece have been properly prepared, proceed with cementation.

4.1 When handling the cement, make sure that there is no light directly pointed at it, as this may accelerate the curing of the cement due to the high light intensity of the reflector.

4.2 Select the color of the cement by comparing it with the VITA color scale or preferably with the Unikk Veneer Try-in color test paste;

4.3 Directly from the syringe or using a spatula, apply a thin layer of cement to the inside of the veneer;

4.4 Adapt the veneer to the tooth and, applying light pressure, remove the excess cement with a disposable micro-applicator or exploratory probe;

4.5 After removing the excess cement and before light-curing, apply glycerin gel to the margins of the restoration to avoid the inhibited layer. Light curing should be carried out for 40 s* on each margin (mesial, distal and cervical) and directly on the vestibular surface.

4.6 Finishing and polishing can be performed using an extra-fine sandpaper disc (on the margins), sandpaper strips (proximal surfaces), polishing pastes and felt discs (on the prosthetic piece).

* Considering a light curing unit with a minimum power of 600 mW/cm² and light emission in the blue spectrum (400 to 500 nm).

Colors

A1, A2, A3.5, B1, BL2 and T (clear)

LV (Low Viscosity) is A2

Presentation forms

Kit

6 syringes (2.5 g each) with 12 Unikk Veneer LC application tips in the colors: A1, A2, A3.5, B1, BL2 and T plus 1 LV syringe.

Refill

1 syringe (2.5 g) with 3 applicator tips of Unikk Veneer LC or Unikk Veneer Try-in in the colors: A1, A2, A3.5, B1, BL2 and T or 1 LV syringe (2.5 g).

Tip packaging

1 package containing 20 application tips.

Basic composition:

Cement: Contains methacrylate monomers, stabilizers, photoinitiator, co-initiator, fillers, silane and pigments.

Try-in: Glycerin, water, pigments and fillers.

Expiration date:

Printed on the product packaging.

Storage conditions:

The product must remain in its original packaging.

Keep the product in a cool place with the packaging always tightly closed.

Protect from direct sunlight.

Store the product at temperatures from 5 °C to 27 °C / 41 °F to 80 °F.

Do not freeze the product.

Do not use the product after the expiration date.

If necessary, with the aid of alcohol, remnants of the unpolymerized product can be cleaned.

Handling Conditions

- The product is indicated for professional use only.
- Carefully read the indications and instructions for use (see QR Code).
- Do not use the product if it is out of date.
- To dispose of the product, follow the legislation of your country.
- Clean the instruments with alcohol after use.
- Replace the cap on the syringe after each use.
- The use of gloves is recommended.
- The capsule presentation is for single use; discard the capsule after use.
- If there is pulp exposure, protect the area with a lining material or base;
- Do not use eugenol-based temporary cements before Unikk Veneer LC;
- The cement is compatible with all classes of adhesives;
- Light cure the cement for at least 40 seconds on each margin and also directly on the vestibular surface, with a light curing unit (halogen or LED light) with a minimum power of 600 mW/cm² or more and light in the blue spectrum (400 to 500 nm). Wear protective glasses when operating the light curing unit.

Warnings

Do not use the product if it is out of date.

Keep out of the reach of children.

Precautions

Professional use only

- This product is indicated only according to its instructions for use.
- Avoid the use of eugenol based lining or provisional materials, as these interfere with the polymerization of the material.
- Avoid contact of the unpolymerized product with the skin, mucosa and eyes.
- Avoid use in patients with a history of allergy to composite resins based on methacrylates. In case of allergy, discontinue use and refer the patient for medical assistance.
- When using a light curing device, eye protection must be used.
- Sensitive to light. Always keep the product closed when it is not in use. After extruding the product out of the syringe or capsule, the contents should be used quickly.
- Light curing equipment must have its power checked with a radiometer frequently.

Contraindications and adverse effects

The product is contraindicated for patients allergic to methacrylates. The product contains methacrylate monomers that can cause mild allergic or irritant reactions in patients sensitive to these substances. In case of allergic reactions to the product, use must be suspended.

This material has been manufactured for dental use only and must be handled, according to the instructions for usage. The manufacturer is not responsible for any damage caused by improper use or manipulation of the product. The user is obliged to check, before use and under his responsibility, if this material is compatible with the intended use, especially when it is not indicated in this instruction for use. Data descriptions are not a guarantee of any kind and therefore are not binding.

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