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Fabricado E Distribuído por

MEDICAL DEVICE INSTRUCTIONS FOR USE

Technical Name: Dental Acrylic Resin

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contato@smartdent.com.br

PRODUCT DESCRIPTION

These are acrylic resins for exclusive use by 3D printers and do not come into direct contact with the patient's skin and mucous membranes. Its working principle is photopolymerization when subjected to wavelength radiation in the range of 395-405 nm.

MODELS

- | | |
|-------------------------------------|---------------------------------------|
| • Smart Print Bio Stain - White | • Smart Print Bio Clear |
| • Smart Print Bio Stain - Black | • Smart Print Bio Clear Guide |
| • Smart Print Bio Stain - Violet | • Smart Print Bio Denture (Cor: Rosa) |
| • Smart Print Bio Stain - A | • Smart Print Bio Denture |
| • Smart Print Bio Stain - B | • Smart Print Bio GoWhite |
| • Smart Print Bio Stain - Blue | • Smart Print Bio Hybrid A1 |
| • Smart Print Bio Stain - Grey | • Smart Print Bio Hybrid A2 |
| • Smart Print Bio Stain - Red | • Smart Print Bio Hybrid A3 |
| • Smart Print Bio Bite Splint Clear | • Smart Print Bio Hybrid B1 |
| • Smart Print Bio Bite Splint +Flex | • Smart Print Bio Hybrid B2 |
| • Smart Print Bio Bite Splint | • Smart Print Bio Hybrid B3 |
| • Smart Print Bio Temp B1 | • Smart Print Bio Hybrid BL1 |
| • Smart Print Bio Temp BL | • Smart Print Bio Hybrid BL2 |
| • Smart Print Bio Try In Calcinável | • Smart Print Bio Hybrid BL3 |

MEDICAL DEVICE INSTRUCTIONS FOR USE

COMPOSITION

Monomers, oligomers, photoinitiators, pigments, stabilizers.

INDICATION, PURPOSE OR INTENDED USE OF THE MEDICAL DEVICE

All models in the Smart Print Bio line are biocompatible. The purpose of each template is described below:

- The **Smart Print Bio Hybrid** models are used in the manufacture of crowns, bridges (up to 3 elements) and other temporary elements. This resin exhibits high hardness and abrasion resistance and has excellent finishing characteristics.
- The **Smart Print Bio Try In Calcinable** model is used for the printing of calcinable elements, presenting excellent definition, resolution, low shrinkage and repeatability, providing easy combustion and no residues.
- Smart Print Bio Denture models are used in the printing of all types of removable denture bases.
- The **Smart Print Bio Clear Guide** model is suitable for the production of surgical guides for implants.
- The **Smart Print Bio Clear models; Smart Print Bio Stain** is intended for the pigmentation of printed and milled provisionals.
- The **Smart Print Bio Bite Splint Clear** and **Smart Print Bio Bite Splint +Flex** models are resins for the manufacture of myorelaxant plaques and translucent bruxism plaques.

WARNING AND PRECAUTIONS

This product should not be used for any application other than as reported.

The product must be handled by a specialized professional;

The use of personal protective equipment is mandatory when handling this product;

Any use outside of the instructions and expiration date is not guaranteed according to the specifications;

Causes eye irritation;

Causes skin irritation;

It can cause an allergic skin reaction;

It can cause irritation of the respiratory tract;

It can cause damage to fertility;

KEEP OUT OF REACH OF CHILDREN AND ANIMALS.

IN CASE OF CONTACT WITH EYES: Rinse thoroughly with water for several minutes. For contact lenses, remove them if it's easy. Continue rinsing. Consult a doctor if irritation occurs.

IN CASE OF CONTACT WITH SKIN: Wash with plenty of soap and water. In case of irritation or rash, consult a doctor. Remove all contaminated clothing and wash it before using it again.

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(16) 3415-0530 | (16) 3419-4735

www.smartdent.com.br

MEDICAL DEVICE INSTRUCTIONS FOR USE

IN CASE OF INHALATION: Go to a ventilated place. See a doctor if you are not feeling well.

IF INGESTION: Rinse mouth with water. Do not induce vomiting. Never give anything orally to an unconscious person. Seek medical help.

IN CASE OF FIRE: Use chemical foam, carbon dioxide, dry powder, or water spray as a means of extinguishing.

PRODUCT PRESENTATION

The product is sold in high-density polyethylene (HDPE) bottles containing 250 g, 500 g and 1 kg of liquid resin. Each package displays a self-adhesive label with all the relevant information for the product (Validity, Batch, Anvisa Registration, Instructions, etc.) They can display a cardboard box, filled with bubble wrap to protect the bottles from bumps and falls, as secondary packaging.

STORAGE AND TRANSPORT

The product should be stored in its original unopened bottle until the time of use. It should be kept protected from light and heat, between 15 °C and 30 °C, in a clean and dry place, where there is no risk of breakage. The product must be transported in the original closed packaging, protected from light and heat. Avoid pressing and shaking the package during transportation, as well as collision with hard objects. Check that the package is intact and properly closed before transport.

PRODUCT DISPOSAL

The product must be disposed of in accordance with the environmental guidelines established by Anvisa's RDC 222/2018.

MEDICAL DEVICE INSTRUCTIONS FOR USE

HOW TO USE THE PRODUCT

1. Handle the product with gloves and eye protection. Pour the amount recommended by the printer manufacturer into the printer tank and start printing;
2. After printing is complete, place the printed parts in an ultrasonic vat containing ethyl alcohol or isopropyl alcohol for 3 to 5 minutes;
3. After washing, dry the parts completely. Pressurized air can be used;
4. Place the parts in the ultraviolet light chamber for subsequent curing. This step can range from 5 to 60 minutes, depending on the resin;
5. After printing, remove the resin remnants from the tank;

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