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Technical Name: Dental Acrylic Resin

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CREA/SP: 5061452320

PRODUCT DESCRIPTION

This product consists of acrylic resins intended exclusively for use in 3D printers and does not come into direct contact with the patient's skin or mucous membranes. Its mechanism of action is based on photopolymerization when exposed to light radiation in the wavelength range of 395–405 nm.

MODELS

Smart Print Model Plus

COMPOSITION

Monomers, Oligomers, Photoinitiators, Pigments, Stabilizers.

INDICATION, PURPOSE, OR INTENDED USE OF THE MEDICAL PRODUCT

Smart Print Model is a line of resins designed for the manufacturing of dental models, used in the production of orthodontic aligners and occlusal splints. These resins are for exclusive use in 3D printers and do not come into direct contact with the patient's skin or mucous membranes.

WARNINGS AND PRECAUTIONS

This product must not be used for any purpose other than that described.

The product must be handled by trained professionals.

The use of personal protective equipment (PPE) is mandatory when handling this product.

Any use beyond the specified indications or expiration period is not guaranteed according to specifications.

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(16) 3415-0530 | (16) 3419-4735

www.smartdent.com.br

- Flammable liquid.
- Causes serious eye irritation.
- Causes skin irritation.
- May cause an allergic skin reaction.
- May cause respiratory irritation.
- May damage fertility.
- Keep out of reach of children and animals.

IN CASE OF EYE CONTACT: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. Seek medical advice if irritation persists.

IN CASE OF SKIN CONTACT: Wash thoroughly with plenty of water and soap. If irritation or rash occurs, seek medical advice. Remove contaminated clothing and wash before reuse.

IN CASE OF INHALATION: Move to fresh air. Seek medical advice if feeling unwell.

IN CASE OF INGESTION: Rinse mouth with water. Do not induce vomiting. Never give anything by mouth to an unconscious person. Seek medical assistance.

IN CASE OF FIRE: Use foam, carbon dioxide (CO₂), dry chemical powder, or water spray for extinguishing.

PRODUCT PRESENTATION

The product is sold in high-density polyethylene (HDPE) bottles containing 250 g, 500 g, or 1 kg of liquid resin. Each package bears a self-adhesive label with all relevant information (shelf life, batch number, instructions, etc.). As secondary packaging, a cardboard box filled with bubble wrap may be used to protect the bottles from impact and falls.

STORAGE AND TRANSPORT

The product must be stored in its original sealed container until use. Keep protected from light and heat, between 15 °C and 30 °C, in a clean, dry place with no risk of breakage.

Transport only in the original closed packaging, protected from light and heat. Avoid pressing or shaking the package during transport and prevent impact with hard objects. Ensure the package is intact and properly sealed before shipping.

PRODUCT DISPOSAL

The product must be disposed of in accordance with the applicable local regulations for chemical and medical waste, ensuring proper environmental protection and the safety of individuals.

INSTRUCTIONS FOR USE

- 1- Handle the product wearing gloves and eye protection. Pour into the printer tank the amount recommended by the printer manufacturer and start printing.
- 2- After printing, place the printed parts in an ultrasonic bath containing ethyl or isopropyl alcohol for 3 to 5 minutes.
- 3- After washing, dry the parts completely (compressed air may be used).
- 4- Place the parts in a UV curing chamber for post-curing. The curing time may vary from 5 to 60 minutes, depending on the geometry and thickness of the printed parts.
- 5- After printing, remove any remaining resin from the printer tank.

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