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Fabricado E Distribuído por

MEDICAL DEVICE INSTRUCTIONS FOR USE

Technical Name: Dental Acrylic Resin

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contato@smartdent.com.br

PRODUCT DESCRIPTION

It is an acrylic resin for exclusive use by 3D printers. Its operating principle is photopolymerization when subjected to wavelength radiation in the range of 395-405nm.

MODELS

Smart Print Bio Vitality

COMPOSITION

Monomers, Oligomers, Photoinitiators, Pigments, Inorganic Fillers, Stabilizers.

INDICATION, PURPOSE OR INTENDED USE OF THE MEDICAL PRODUCT

Smart Print Bio Vitality is used for printing crowns and elements to remain in the mouth for long periods (up to 10 years). This resin is biocompatible and for exclusive use by 3D printers.

WARNING AND PRECAUTIONS

This product should not be used for any other application than that reported.

The product must be handled by a specialized professional;

The use of personal protective equipment is mandatory when handling this product;

Any use outside the indications and expiration date is not guaranteed according to the specifications;

Flammable liquid;

Causes eye irritation;

Causes skin irritation;

It can cause a skin allergic reaction;

MEDICAL DEVICE INSTRUCTIONS FOR USE

It can cause irritation of the respiratory tract;

It can cause damage to fertility;

KEEP OUT OF REACH OF CHILDREN AND ANIMALS.

IN CASE OF CONTACT WITH EYES: Rinse thoroughly with water for several minutes. In the case of contact lens wear, remove them if it is easy. Continue rinsing. Consult a doctor if irritation occurs.

IN CASE OF CONTACT WITH SKIN: Wash with plenty of soap and water. In case of skin irritation or rash, consult a doctor. Remove all contaminated clothing and wash it before wearing it again.

IN CASE OF INHALATION: Go to a ventilated place. See a doctor if you feel unwell.

IF SWALLOWED: Rinse mouth with water. Do not induce vomiting. Never give anything orally to an unconscious person. Seek medical help.

IN CASE OF FIRE: Use chemical foam, carbon dioxide, dry powder or water spray as a means of extinguishing.

PRODUCT PRESENTATION

The product is sold in high-density polyethylene (HDPE) bottles containing 250g, 500g and 1Kg of liquid resin. Each package displays a self-adhesive label with all the information relevant to the product (Validity, Lot, Anvisa Reg., Instructions, etc.) They may display as secondary packaging a cardboard box, filled with bubble wrap in order to protect the bottles against shocks and falls.

STORAGE AND TRANSPORTATION

The product should be stored in its original unopened bottle until the time of use. It should be kept protected from light and heat, between 15 °C and 35 °C, in a clean, dry place, where there is no risk of breakage. The product must be transported in the original closed packaging, protected from light and heat. Avoid pressing and shaking the package during transport, as well as collision with hard objects. Check that the package is intact and correctly closed before transport.

PRODUCT DISPOSAL

The product must be disposed of in compliance with the environmental guidelines established by local legislation.

MEDICAL DEVICE INSTRUCTIONS FOR USE

HOW TO USE THE PRODUCT

1. Handle the product with gloves and eye protection. Pour the amount recommended by the printer manufacturer into the printer tank and start printing;
 2. After printing is finished, place the printed parts in an ultrasonic vat containing ethyl alcohol or isopropyl alcohol for 3 to 5 minutes;
 3. After washing, dry the parts completely. Pressurized air can be used;
 4. Put the parts into UV light chamber for post-curing. This step can range from 5 to 60 minutes, depending on the resin;
 5. After printing, remove the resin remnants from the tank;
- For more information, visit: <http://smartdent.com.br/manuais>

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